Instrument Tray / 器械托盘 / 器械托盤 / Podložak za instrumente / Tácek na nástroje /
Instrumentbakke / Instrumententray / Instrumenttitarjotin / Plateau à instruments /
Instrumentenschale / Vassoio per strumenti / 機器トレイ / Taca na narzędzia /
Bandeja de instrumentos / Bandeja de instrumentos / Лоток для инструментов /
Bandejas de instrumentos / Instrumentbricka

Rx ONLY

Rx ONLY Federal (USA) law restricts this device to sale by or on the order of a physician or practitioner

Indications For Use

Instrument trays are intended to provide a general configuration to organize, store, protect and transport medical devices for use in surgical procedures.

Precautions

If you choose to use the instrument tray as part of a sterilization system for reprocessing devices:

- Ensure that you validate the tray within your internal processes and systems.
- Follow all instructions for sterilization of the device(s) you are reprocessing.
- Gravity Steam sterilization must not be used for instrument trays as cycle times would be exceedingly long.
- Use prevacuum cycles only for steam sterilization.
- STERRAD processing must not be used for Xomed instrument trays.
- Instrument Trays are not intended to provide a sterile barrier to prevent re-contamination of sterilized instruments; and must be double wrapped in FDA-cleared hospital CSR wrap for this process to maintain the sterility of the instruments contained inside the tray.
- Remove instruments, loose internal tray components and lid prior to cleaning instruments and/or tray.
- When applying dry times to Integra LifeSciences instrument trays and their accessories, dry times outside the standard healthcare prevacuum parameters may be required.

Drying Time

- Instruments Only and Aluminum/Stainless Steel Instrument Trays : 15 to 30 minutes and visual inspection for dryness.

- Polymer-based (plastic) Instrument Trays: Minimum drying time 40 minutes and visual inspection for dryness

Contraindication

Instruments should not be used for anything other than their intended use.

Warning

DO NOT flash sterilize the Instrument Trays. The instruments have not been validated for flash sterilization.

Discard instrument after suspected Creutzfeldt-Jakob Disease (CJD) exposure; the Instrument Trays has not been validated to withstand the chemical and thermal exposures recommended to eradicate prions.

Adverse Reactions

None known

Pre-Use, Handling, and Inspection of Instruments

- Read the Instructions for Use and keep them in a safe place.
- Use the product only in accordance with its intended use, see Indications.
- Use of an instrument for a task other than that for which it is intended could result in a damaged or broken instrument, or one which provides an unsatisfactory performance.
- Instruments should be handled and operated by personnel completely familiar with their use, assembly, and disassembly.
- Instruments must be rendered safe for handling, inspection, and assembly by wearing appropriate personal protection equipment (PPE) as promulgated by OSHA and AORN.
- Inappropriate use of instruments will lead to damage that is usually not repairable.

- Instruments must be thoroughly inspected upon receipt and prior to use to assure proper functioning. Failure to make a complete inspection to assure proper operation and function of instrument may result in unsatisfactory performance.
- Instrument Trays are supplied non-sterile and must be pre-cleaned, cleaned, visually examined, lubricated, and sterilized prior to use. Please see pre-cleaning, cleaning, visual examination, lubrication, and sterilization procedures below.
- Check screws on instruments after ultrasonic cleaning. Vibration from ultrasonic cleaning may cause them to loosen or fall out.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective.
- Replace any damaged components immediately with original spare parts.
- Store products in a dry, clean and safe place

Automated Cleaning

All trays shall be thoroughly rinsed with tap water prior to placement in the automatic washer.

Remove instruments and equipment from any instrument trays before placing into washer baskets. Orient devices following recommendations of washer/disinfector manufacturers.

Use neutral pH detergent recommended by washer/disinfector or detergent manufacturers.

PHASE	RECIRCULATION TIME	WATER TEMPERATURE	DETERGENT TYPE
Pre-wash 1	3 minutes	cold tap water	N/A

Enzyme Wash	5 minutes	hot tap water	neutral pH enzymatic detergent
Wash 1	5 minutes	66°C set point	neutral pH detergent
Rinse 1	3 minutes	hot tap water	N/A

Manual Cleaning

Soak in lukewarm (less than 43°C), mild (pH 7.0 - 8.5), enzymatic detergent and deionized water for a minimum of 2 minutes. Use a soft instrument brush to scrub instrument trays while submerged in cleaning solution to remove organic matter. Rinse with deionized water, then clean ultrasonically in a lukewarm (less than 43°C), mild (pH 7.0 - 8.5), detergent and deionized water for 10 minutes.

Rinse thoroughly with deionized water. Clean in this manner until no visible soil remains on the instrument tray.

Dry with compressed air, or wipe dry with a lint-free cloth. Examine instrument trays for any staining or deterioration; remove from use as appropriate.

Note: When using an ultrasonic cleaner or a spray washing machine, follow the manufacturer's recommendations for positioning of instrument trays. Following cleaning, lightly lubricate instrument trays with movable parts. Use a lubricant intended for sterilizable instruments such as a water-soluble instrument milk. Do not use silicone spray.

Prevacuum Only

Use prevacuum cycles only for steam sterilization.

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Integra and the Integra logo are registered trademarks of Integra LifeSciences Corporation or its subsidiaries in the United States and/or other countries.
©2015 Integra LifeSciences Corporation. All Rights Reserved.
Integra York PA, Inc.
Integra LifeSciences Services (France) SAS