

Rx ONLY

Caution: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician or practitioner



Consult Instructions for Use


Chirurgie Laparoscopique Instruments démontables / Laparoscopic surgical instruments, modular /

Strumenti chirurgici per laparoscopia, modulari / Laparoskopische chirurgische Instrumente, modular / Instrumental quirúrgico desmontable para cirugía laparoscópica / Laparoskopische instrumenten, demontabel / Laparoskopiske kirurgiske instrumenter, modulære / Kirurgisen laparoskopian modulaarisia instrumentteja // Laparoskopiska kirurgiska instrument, modulär / Instrumentos cirúrgicos laparoscópicos desmontáveis / Λαπαροσκοπικά χειρουργικά όργανα, συναρμολογούμενα / Instrumenty chirurgiczne do laparoskopii, modularne / Laparoskopické chirurgické nástroje, modulární / Laparoskopos sebészeti műszerek, moduláris / Laparoskopik cerrahi aletler, modüler / Instrumenter til laparoskopisk kirurgi, modulære

1.1. Montage Instruments démontables rotatifs / Rotative modular instrument assembly / Montaggio strumenti a rotazione modulari /

Drehbares, modulares Instrument / Montaje de instrumental rotatorio desmontable / Montage van het draaibare opzetstuk / Drejelig modulær instrumentdel / Kierrettävän modulaarisen instrumentin kokoaminen / Roterande modulär instrumentmontering / Montagem de instrumentos rotativos desmontáveis / Συναρμολόγηση περιστροφικών συναρμολογούμενων οργάνων / Zmontowanie obrotowego instrumentu modularnego / Sestavení rotačních modulárních instrumentů / Forgatható moduláris készülékgyűjtés / Döner modüler aletin takılması / Montering av roterende modulært instrument


Mors / Jaw / Morsetto / Backe / Boca / Bek / Kæbe / Leuat / Käft / Garra / Σιαγόνα / Szczeka / Pracovní nástavec / Pofa / Ağiz / Kjeve

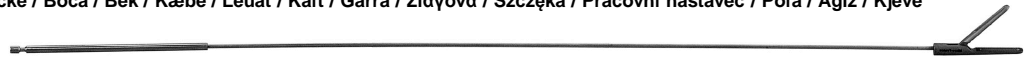

Tube / Tube / Tubo / Rohr / Tubo / Buis / Rør / Putki / Rör / Rør / σωλήνας / Tubo / Tüp / Trubice / Cső / Tüp / Rør







Poignée / Handle / Impugnatura / Griff / Mango / Handgriep / Håndtag / Kahva / Handtag / Cabo / Λαβή / Rączka / Násada / Fogantyú / Tutamak / Håndtak

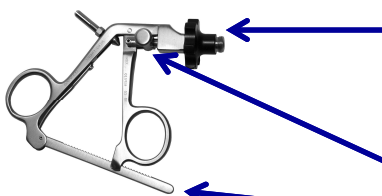

Volant de rotation / Rotation wheel / Rotellina di rotazione / Drehrad / Rueda giratoria / Rotatiewiel / Drejhjul / Kiertopyörä / Rotationshjul / Revólver rotativo / Τροχός περιστροφής / Pokrętko / Otočné kolečko / Forgókerék / Tekerlek dönme / Hjulrotasjon

Cliquet / Locking button / Pulsante di blocco / Arretiertaste / Botón de bloqueo / Vergrendelingsknop / Låseknapp / Lukituspainike / Låsknapp / Botão de bloqueio / Κουμπί εμπλοκής / Zapadka / Západka / Zároló billentyű / Kilit düğmesi / Låseknapp

Crémaillère / Ratchet / Dente di arresto / Rotationssperre / Cremallera / Vergrendelingspal / Lås / Salpa / Spärrhake / Lingueta / Οδοντωτός κανόνας / Szyna / Kolejnička / Retesz / Circir / Ratchet

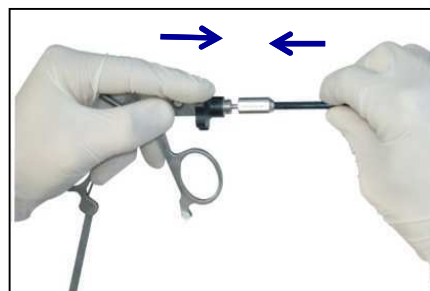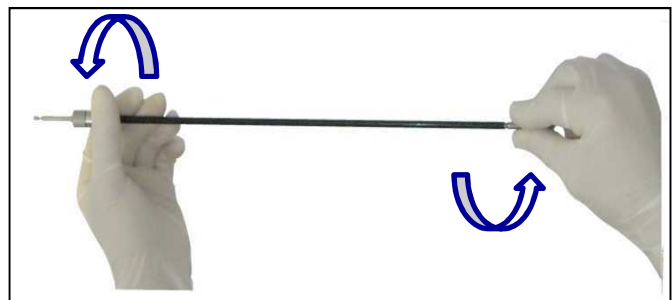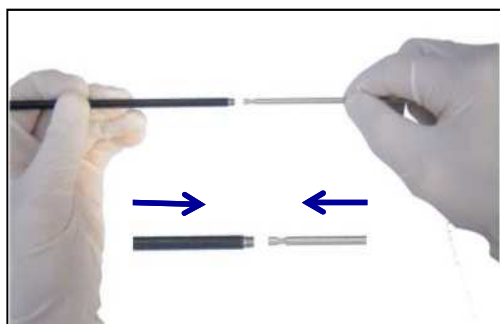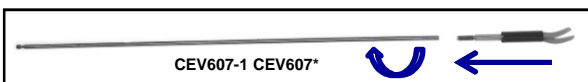
 Manufacturer:
 Integra MicroFrance SAS
 Le Pavillon • 03160 Saint Aubin Le Monial • France
 integrallife.com


Product complies with the requirements of directive 93/42/EEC





Rx ONLY

Caution: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician or practitioner




Consult Instructions for Use

1.2. Démontage Instruments démontables rotatifs / Rotative modular instrument disassembly / Smontaggio strumenti a rotazione modulare / Demontage des drehbaren, modularen Instruments / Desmontaje de instrumental rotatorio desmontable / Demontage van het draaibare opzetstuk / Drejelig modulær adskillelig instrumentdel / Kierrettävän koottavan instrumentin purkaminen / Roterande modulær instrumentdemontering / Desmontagem de instrumentos rotativos desmontáveis / Αποσυναρμολόγηση περιστρεφόμενων συναρμολογούμενων οργάνων / Rozmontowanie obrotowego instrumentu modularnego / Sestava rotačních modulárních instrumentů / Forgatható moduláris készülékgység / Döner olmayan modüler aletin çıkarılması / Demoning av ikke-roterende modulært instrument

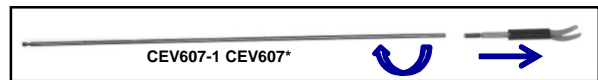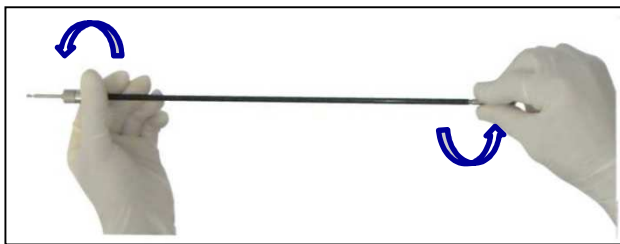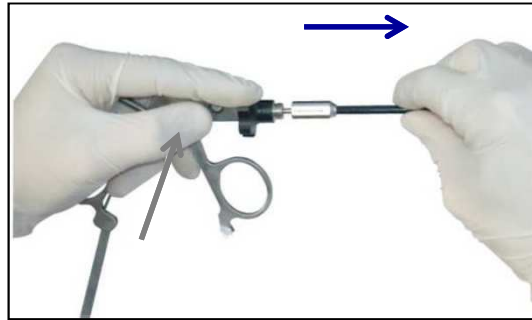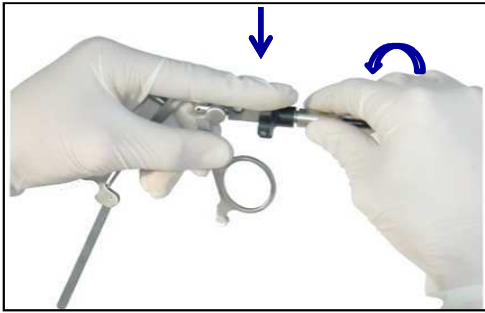

2.1. Montage Instruments démontables non rotatifs / Non-rotative modular instrument assembly / Montaggio strumenti non a rotazione modulare / Nicht drehbares, modulares Instrument / Montaje de instrumental no rotatorio desmontable / Montage van het niet-draaibare opzetstuk / Ikke-drejelig modulær instrumentdel / Ei-kierrettävän purettavan instrumentin kokoaminen / Ikke-roterande modulær instrumentmontering / Montagem de instrumentos não rotativos desmontáveis / Συναρμολόγηση μη περιστρεφόμενων συναρμολογούμενων οργάνων / Zmontowanie nieobrotowego instrumentu modularnego / Sestavení nerotačních modulárních instrumentů / Nem forgatható moduláris készülékgység / Döner olmayan modüler aletin takılması / Montering av ikke-roterende modulært instrument

Mors / Jaw / Morsetto / Backe / Boca / Bek / Kæbe / Leuat / Käft / Garra / Σιαγόνα / Szczeka / Pracovní nástavec / Pofa / Ağız / Kjeve

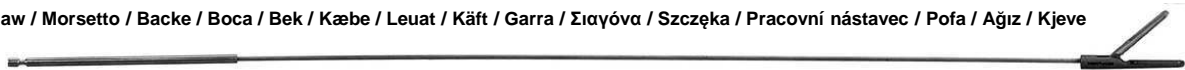

Poignée / Handle / Impugnatura / Griff / Mango / Handgreep / Håndtag / Kahva / Handtag / Cabo / Λαβή / Rączka / Násada / Fogantyú / Tutamak / Håndtak



Tube / Tube / Tubo / Rohr / Tubo / Buis / Rør / Putki / Rör / Rør / σωλήνας / Tubo / Tüp / Trubice / Cső / Tüp / Rør

Molette de blocage / Locking Knob / Manopola di blocco / Arretierhebel / Botón de bloqueo / Vergrendelingsknop / Låseskrue / Lukitusosa / Låsskruv / Botão de fixação / Ασφάλεια / Gałka unieruchamiająca / Zamykací točítko / Rögzítőcsavar / Kilitleme düğmesi / Låseknapp


 Manufacturer:
 Integra MicroFrance SAS
 Le Pavillon • 03160 Saint Aubin Le Monial • France
 integralife.com



Product complies with the requirements of directive 93/42/EEC

Rx ONLY

Caution: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician or practitioner



Consult Instructions for Use

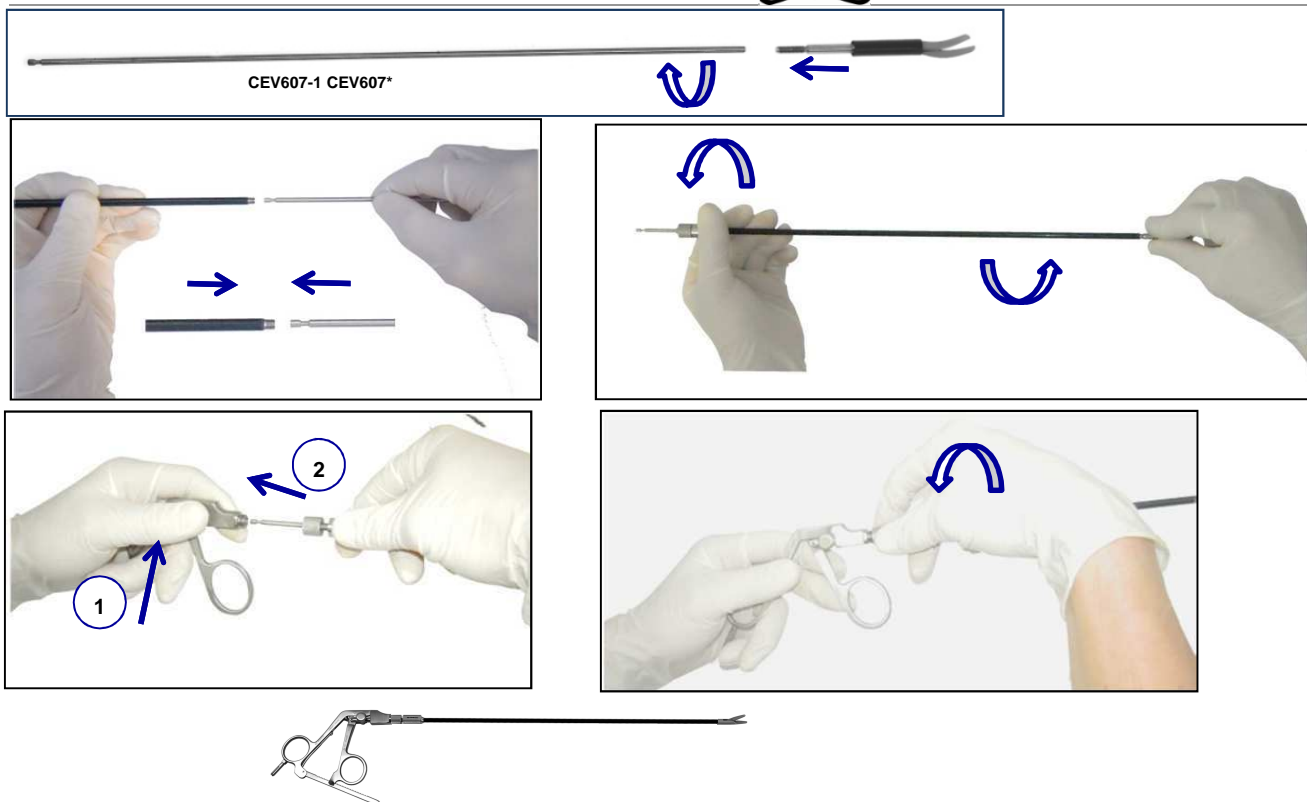

2.2. Démontage Instruments démontables non rotatifs / Non-rotative modular instrument disassembly / Smontaggio strumenti non a rotazione modulari / Demontage des nichtdrehbaren, modularen Instruments / Desmontaje de instrumental no rotatorio desmontable / Demontage van het niet-draaibare opzetstuk / Ikke-drejelig modulær adskillelig instrumentdel / Eikierrettävän koottavan instrumentin purkaminen / Ikke-roterende modulær instrumentdemontering / Desmontagem de instrumentos não rotativos desmontáveis / Αποσυναρμολόγηση μη περιστρεφόμενων συναρμολογούμενων οργάνων / Rozmontowanie nieobrotowego instrumentu modularnego / Sestava nerotačnich modulárnich instrumentů / Nem forgatható moduláris készülékesség / Döner olmayan modüler aletin çıkarılması / Demontering av ikke-roterende modulært instrument

